


TributeWrap™ NEW Ease and comfort within your reach.

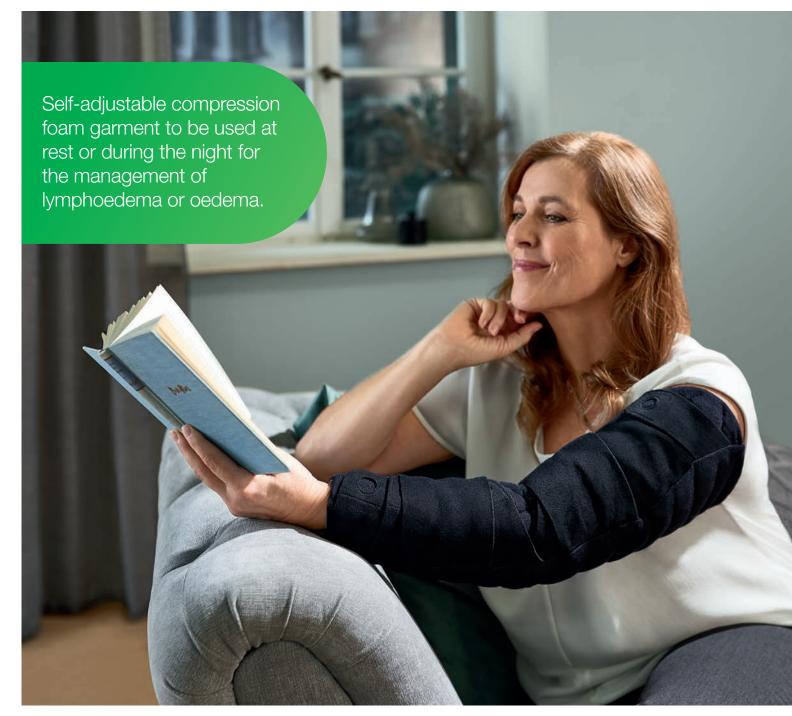

TributeWrap™ Promoting skin health with proven foam technology.

TributeWrap is an adjustable compression garment with foam technology that nestles into the tissue to promote skin health¹. It supports easy self-management of lymphoedema and oedema during low activity by comfortably adjusting to your patient's unique shape and lifestyle.

TributeWrap helps maintain gains made during therapy, while allowing the flexibility to support further reduction. Simply slide it on, adjust the straps, and cover with Sleep Sleeve for comfortable evening compression.



Off-the-Shelf



Evening & Nighttime



Low Activity

Explore the other great benefits of TributeWrap >



Easy-to-use finger grip on every strap.
Simple adjustability for the right fit and contour.





Sleep Sleeve
Complimentary
cover included
with all styles to
ensure the straps
don't get tangled
in the sheets.



TributeWrap™ Wrist to Axilla



Chevron channeling

Designed based on MLD principles of directing fluid to collateral pathways.



Adjustable strapping system

Easily conforms for the right, comfortable fit, and has flexibility to support further reduction.



Supports mobility

Angled straps at flexion points for a comfortable and functional design.



6-month warranty

TributeWrap is guaranteed to be free of material and manufacturing defects for the six-month life of the garment.



"In the morning, I took it off and couldn't believe the difference! I wondered if this was a fluke, but I've noticed the same results every morning!

This is outstanding! It is easier to put on my daytime flat knit garments, and my legs have less fatigue throughout the day.

I am convinced TributeWrap does make a significant difference!"

John, Lymphoedema patient



TributeWrap[™] NEW Self-adjustable compression foam garment.

Indications

- mild to severe lymphoedema, stage 1-3
- chronic oedema (non-blood related)

Precautions

- impaired sensation (peripheral neuropathy)
- cognitive impairment
- any open wounds should be dressed prior to application

Contraindications

- moderate to severe peripheral arterial disease
- arterial insufficiency or ABPI < 0.8
- untreated congestive heart failure
- untreated infection (e.g. cellulitis)
- allergies or sensitivities to materials
- acute DVT
- severe cognitive impairment

For more information, please call:

Complete Medical Compression WA

P: 08 6174 0314 M: 0425124 576

E: sales@cmompression.com.au

TributeWrap Glove

Self-adjustable compression foam garment for hand and fingers

| Size | Circumfer | ence (cm) | | | |
|------|----------------------|-----------|-------|---------|--|
| | Palm (cB) Wrist (cC) | | | REF | |
| S | 16 – 20 | 16.5 – 21 | Left | 146 783 | |
| | | | Right | 146 786 | |
| М | 18 – 22 | 17.5 – 22 | Left | 146 784 | |
| | | | Right | 146 787 | |
| L | 20 – 24.5 | 18.5 – 23 | Left | 146 785 | |
| | | | Right | 146 788 | |

Not made with natural rubber latex



TributeWrap Arm

Self-adjustable compression foam garment for wrist to axilla

| Size | Circumference (cm) | | | Length (cm) | | |
|------|--------------------|---------------|----------------|-------------|-------|---------|
| | Wrist (cC) | Elbow (cE) | Axilla (cG) | Arm (IG) | REF | |
| S | 16 – 21 | 22 – 28 | 25 – 31 | 43 – 47 | Left | 146 777 |
| 3 | 10 – 21 | 22 – 20 | 20-31 | | Right | 146 780 |
| М | 17 – 22 | 26 – 32 | 30 – 37 | | Left | 146 778 |
| | | | | | Right | 146 781 |
| L | 18 – 23 | 30 – 36 | 35 – 44 | | Left | 146 779 |
| | | | | | Right | 146 782 |

Not made with natural rubber latex 40°



TributeWrap Below Knee

Self-adjustable compression foam garment for foot and calf

| Size | Circumference (cm) | | Length (cm) | | | |
|------|--------------------|--------------|------------------------|---------|-------|---------|
| | Ankle (cB) | Calf (cC) | Foot Calf (IA) (ID) | | REF | |
| S | 24 – 30 | 30 – 36 | - 19 – 22 - | 40 – 44 | Left | 146 789 |
| 5 | 24 – 30 | 30 – 36 | | | Right | 146 792 |
| М | 27 – 33 | 33 – 41.5 | | | Left | 146 790 |
| | | | | | Right | 146 793 |
| L | 30 – 36 | 36 – 45 | | | Left | 146 791 |
| | | | | | Right | 146 794 |

Not made with natural rubber latex

